

ComfortSeal* Masks with Elbow for use with AEROECLIPSE* BAN* Nebulizers

- ✓ Masks contour gently to the face providing a secure and comfortable fit
- ✓ Soft silicone elastomer to safeguard against facial irritation
- ✓ Anatomically designed for minimal dead space
- ✓ Available in three sizes: small (0-18 months), medium (1-5 years) and large (5+ years)



Manufacturer: Trudell Medical International

Origin: Canada

Ordering Information:

- Part Number: 105503110010 (small); 105502110010 (medium); 105504110010 (large)
- EAN: 0 762860 000629 (small); 0 762860 000636 (medium); 0 762860 000643 (large)
- Packaging (all sizes): Bulk
- Package Contents (all sizes): One **ComfortSeal*** Mask with Elbow Adapter and instructions for use
- Case Quantity (all sizes): 10

Attributes:

- Single patient – multiple use
- For optimum performance replace after 6 months
- Not made or manufactured with bisphenol A (BPA), phthalates, latex or lead
- CE marked
- Product Reference Number (REF): 105503 (small); 105502 (medium); 105504 (large)

Cleaning Method:

- Hand washing

The device can be cleaned after each use. Test results indicate that the methods tested have no adverse effect on the performance of the device.

Refer to the instructions for use for additional information.

Disinfection Methods:^{††}

- 70% isopropyl alcohol
- Bleach
- Milton[†], Control III[†] or Dodie[†]

^{††}NOTE: Bacteria load testing was not performed for these disinfection methods.

The device can be disinfected each day. Test results indicate that the methods tested have no adverse effect on the performance of the device.

Refer to the instructions for use for additional information.

